

# Elpis Ultra

## User's Manual

Before using this device, read these instructions carefully for proper use and to avoid unnecessary trouble. This manual is based on the default settings of the device. Please keep this user manual for future maintenance use.

### Supports:



iOS 8.0 & above



Android 4.4 & up



Bluetooth 4.0

### Appearance:




1. Display Face

2. Home Button

3. Watch Strap

### Charging:

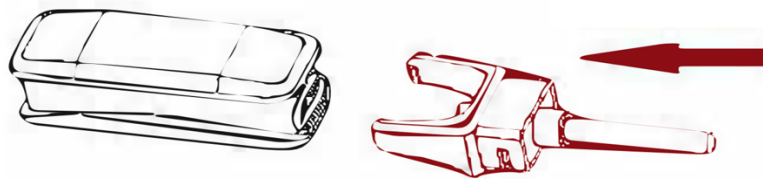

Before using your device, it is wise to fully charge it. It will take about two hours for a full charge. You can charge your device with the charger cable provided in the packaging. With the convenience of it being a USB cable, you can use your computer/laptop or any other USB compatible plug-ins to charge your device. As shown above – detach the main body from the strap and insert it into the prongs. Careful as you detach it after charging.

# Elpis Ultra

## User's Manual

### Start:

Your home button toggles through your functions and features. It also acts as a selection button.

Once your device is fully charged, usually it turns on after it's fully charged, but if not - place your finger on the home button and HOLD until it turns on.

Turn on your 'Bluetooth' on your phone and pair with your device using the app.

Download the App, 'FitCloud' from the App Store or Google Play.

Turn your phone's Bluetooth on to pair with your device.

Open the app and TAP on 'Bind Device' to search and pair your device.

You will know that the pairing is successful if you see the Bluetooth icon on your device is enabled. Make sure your 'Bluetooth' is enabled on your phone. Also make sure that your phone and the wristband are close together and that there are no other competing Bluetooth devices in the vicinity. After pairing is complete, the range between your device and phone is up to 25 feet.

Once pairing is complete, you can toggle through your app to set up each data manually.

### Bracelet Functions:

Time

Heart Rate

Blood Pressure

Blood Oxygen

Steps

Distance

Calories

Sleep Monitoring

Remote Camera

Sedentary Alert

Call Alert

Message/App Alert

Timer

Find Phone



# Elpis Ultra

## User's Manual

While on the main display face (Time, Date...), HOLD the home button until it enters the functions list; TAP to scroll through.

To access, you will have to HOLD the home button on the specific function you would like to open up.

Remember – TAP to scroll through options, HOLD to access.

When you enter the Heart Rate, Blood Pressure, and Blood Oxygen Monitor – make sure your body is relaxed, sitting quietly, and you're wearing your device snugly to the skin on your wrist for optimal and accurate readings.



### Warranty:

Each device bought from Elpis World has a 1-year manufacturer's warranty from the date of purchase.

### Specifications:

Display Screen: 0.66" OLED

Battery Capacity: 50mAh

Battery: Lithium Polymer Battery

Net Weight:  $\approx$ 14.5g

Synchronization: Bluetooth 4.0

Working Temperature: -10°C-50°C

Water Resistant: IP67

Standby Time: up to 10 Days

Work Time: 5-7 Days

# Elpis Ultra

## User's Manual

### Disclaimer:

Although Elpis technology products are not classed as medical devices, our devices do act as an extremely accurate wellness monitor to provide to you instant peace of mind, right from your wrist, in real-time. Elpis technology products certainly do possess technology features & functions that are truly remarkable.

\* Our products are suitable for personal, in-home-use monitoring & self-evaluation for wellness of your heart rate, blood pressure, blood oxygen measurements. The device measuring results are for your own reference and are not to be used for medical diagnosis, medical purpose or for the basis of any treatment.

\* All values are considered ambulatory measurements and are subject to variants with grades in accuracy. If you do experience any abnormal measurements, you are advised to immediately consult your physician and/or medical professional.

\* This device is not a medical device and is not intended to diagnose, treat, cure, detect or prevent any disease.

\* Follow the instructions in your Elpis User Manual carefully, accurately & completely.

\* If your device has a GPS or LBS coordinate feature, data may be inaccurate, incomplete or unavailable due to unforeseen & uncontrollable circumstances.

\* For optimal results, place the device snugly against your skin on your the wrist.

### **WARNING! CHOKING HAZARD!**

Please keep your Elpis technology products away from Children and Pets. Small parts could possibly become a Choking Hazard. Elpis products are **NOT SUITABLE** in any manner for children under the age of 36 months. We recommend you keep an Elpis User Manual for future reference.

### Contents in Box:

Elpis World Device

Charger Cable